HEAVY-DUTY FOLDABLE **SHOWER SEAT** WITH WALL MOUNTED BACKREST

Instruction Manual

Wall-mounted Teak Shower Seat

Manual de instrucciones

Asiento de ducha de teca de pared

EL YOKU

EL YOKU

COMPONENTS INCLUDED

- Teak Shower Seat
- #12 Wood Mounting Screws-Qty 12

TOOLS REQUIRED

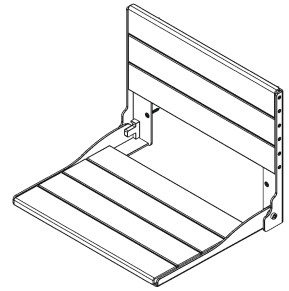
- Pencil
- Leveler
- Stud Finder
- Drill with Phillips bit #3
- Protective Eye-wear

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SAFETY GUIDE

- For safe and secured installation, ensure the seat is mounted into wood studs/backing
- The shower seat should not be installed in fiberglass or acrylic shower enclosures
- Shower seat should be installed by a skilled professional
- Seat can only accommodate one person at a time and has a maximum weight capacity of 1100 lbs

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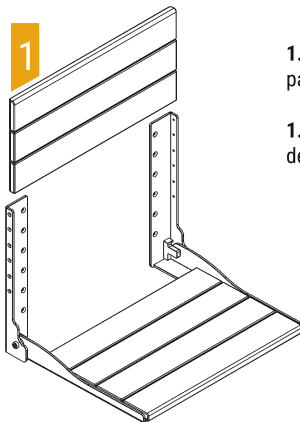
INSTRUCTIONS

Follow the installation instructions carefully. If correct installation procedures are not followed, the seat may become loose, causing serious injury.

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1. To see the top 12 mounting holes, remove the top three back rest panels.

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2. Identify the mounting location of the shower seat making sure the shower seat is level.

2. **A** Be sure the mounting holes are located over studs or wood backing.

2. **B** The wall must be load bearing and strong enough to bear a load of 1100lbs at each of the wall fixings.

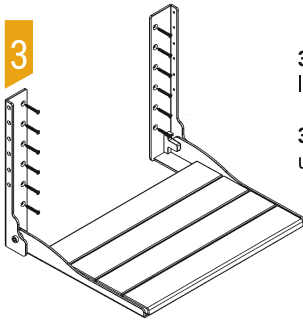
2. **C** Recommended seat height is 15" – 21" from the shower floor.

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3. Once the desired location is determined, mark the 12 mounting hole locations.

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4. Set aside the shower seat and drill a 1/8" pilot hole through the tile in each mounting location. Drill a 1/4" hole through the tile only at the same locations to allow the wood screw to pass through.

4. **A** Once complete, ensure the wall is clear of any debris.

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5. Mount the shower seat to the wall using the 12 wood mounting screws. Start with the top two mounting hole locations.

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6. Once all screws have been installed and tightened, mount / install three back rest panels one after the other with provided screws only.

6. **A** Test the seat to make sure it is installed properly.

6. **B** Do not modify this product in any way. Check the seat periodically after installation to be sure all screws are secured.

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CLEANING INSTRUCTIONS

- Clean the seat with a warm and damp cloth.
- The natural properties of teak wood wither over time when exposed to the environment. If you desire to keep the current look of the wood, treat the teak with certified teak oil.

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LIMITED LIFETIME WARRANTY:

If this product fails due to materials or workmanship at any time during the life of the product, EL YOKU will replace it free of charge, postage-paid. Contact EL YOKU at (860) 317-7100 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. TO THE EXTENT PERMITTED BY LAW, EL YOKU DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

LIMITATION OF LIABILITY:

THIS WARRANTY IS YOUR SOLE REMEDY AND EL YOKU SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOT WITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL EL YOKU'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit www.elyoku.com
Meets or exceed the applicable requirements of the following standards/guidelines:
The Americans with Disabilities Act (ADA)

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